




1. Sondenband aus der Verpackung entnehmen

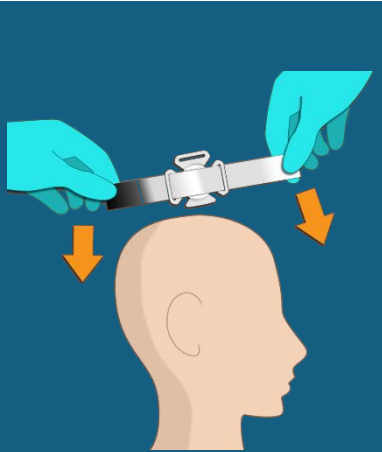


2. Sondenband auf Kopf platzieren

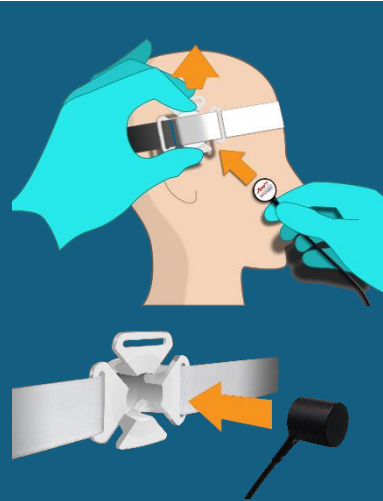


3. Sondenhalterung leicht anheben und die Ultraschallsonde von innen einsetzen. Das Kabel führt im 45°-Winkel aus der Halterung

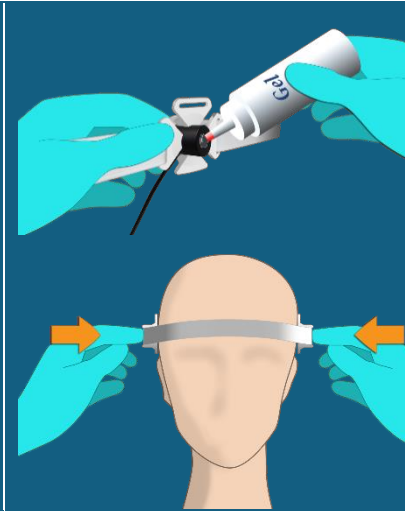


4. Beide Ultraschallsonden mit Ultraschallgel versehen und leicht andrücken

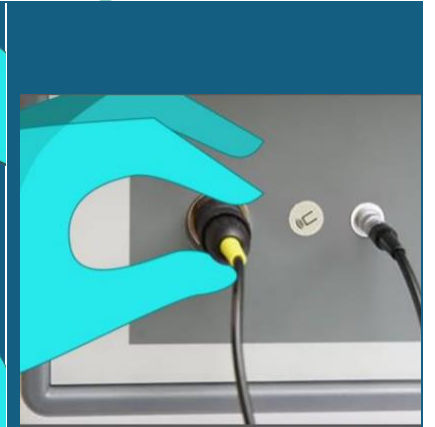


5. Stecker der Ultraschallsonden in Gerät einstecken

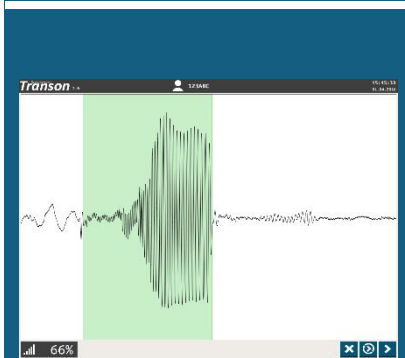

6. Signalinitialisierung von mindestens 70%

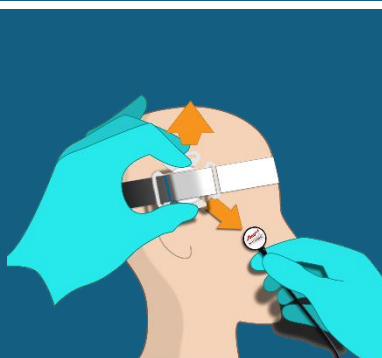


7. Ultraschallsonden nach durchgeführter Messung entfernen

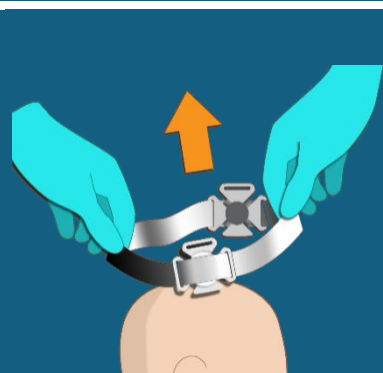

8. Sondenband abnehmen



9. Sondenband mit Sondenhalter verwerfen und Ultraschallsonden reinigen

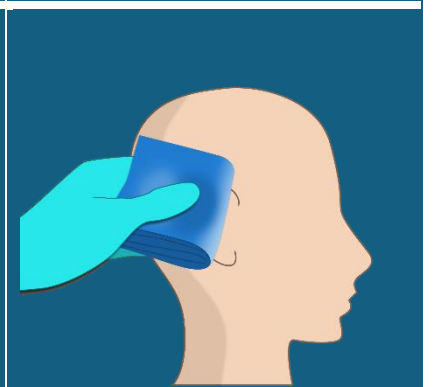

10. Messstellen reinigen

<p>1. Take headband from pouch</p>	<p>2. Place headband on patient's head</p>	<p>3. Lift headband slightly, insert probe with cable facing away from head; cable will protrude from probe holder in 45° angle</p>	<p>4. Cover front of probes with ultrasonic gel and apply light pressure on both of them</p>	<p>5. Insert two plugs into device</p>
<p>6. Wait for signal to initialise above 70%</p>	<p>7. Remove probes after measurement</p>	<p>8. Remove headband</p>	<p>9. Dispose of headband and probe holders, clean probes of gel</p>	<p>10. Wipe contact points on the skull</p>